Primer to enhance bonding between ceramics and resins  
 Instruction for use  
 Primer zur Erhöhung des Haftverbundes zwischen Keramikwerkstoffen und Komposit-Kunststoffen  
 Gebrauchsanleitung  
 Primaire d'adhérence entre les céramiques et les résines  
 Mode d'emploi  
 Primer destinato a creare l'adesione tra ceramica e resina  
 Istruzioni d'uso  
 Adhesivo para mejorar la adhesión entre materiales cerámicos y resinas  
 Instrucciones de uso

**TOKUSO**  
**CERAMIC PRIMER**

CE 0123  
 REF BIS 01/02

Manufacturer  
**Tokuyama Dental Corporation**  
 38-9 Taitou 1-chome, Taitou-ku, Tokyo, Japan

EC Rep.  
**Tokuyama Europe GmbH**  
 Oststrasse 10, 40211 Düsseldorf, Germany  
 Tel: +49 211 1754480 Fax: +49 211 357379

North/South America/Middle East  
**J. MORITA USA, Inc.**  
 9 Mason, Irvine, CA 92618, U. S. A.  
 Tel: (949) 581 9600 Fax: (949) 465 1095  
 Tel: (800) 831 3222  
 Email: info@jmoritusa.com

## ENGLISH

**CONTENTS:**  
 TOKUSO CERAMIC PRIMER A & B each 5mL

### PRODUCT DESCRIPTION AND GENERAL INFORMATION

TOKUSO CERAMIC PRIMER is a primer containing a silane coupling agent to improve the adhesion between ceramics and resins. It can be effectively used as a primer in conjunction with the cementation and repair of vacuum fired, castable and computer generated ceramic restorations.

### CLINICAL APPLICATIONS

- 1) Repair of ceramic materials
- 2) Cementation of ceramic veneers
- 3) Cementation of ceramic inlays or crowns

### CLINICAL PROCEDURES

- 1) Roughen the surface of the restoration with sandblast or grinding with a diamond bur using a slow speed to prepare the area for adhesion.
- 2) Thoroughly rinse and dry the surface. No etching with phosphoric acid is necessary.
- 3) Place a drop of each TOKUSO CERAMIC PRIMER A and B in the mixing well.
- 4) Mix thoroughly and evenly with the disposable brush or the mini sponge.
- 5) Apply the mixed primer on the ceramic surface to be bonded thoroughly. DO NOT over apply.
- 6) Allow the material to stand for 10 seconds prior to bonding.
- 7) Proceed with the bonding procedure according to the manufacturer's instruction for the use of the adhesive material.

### PRECAUTIONS

- 1) When using TOKUSO CERAMIC PRIMER for cementing ceramic restorations, be sure to confirm whether or not the restorations are made of Alumina and/or Zirconia. DO NOT APPLY TOKUSO CERAMIC PRIMER for Alumina and/or Zirconia restorations. It will adversely affect optimum cementation of these restorations. For Alumina and/or zirconia restorations, simply roughen restoration surfaces by using a sandblaster or diamond bur before cementing.
- 2) Use TOKUSO CERAMIC PRIMER only as directed. DO NOT use TOKUSO CERAMIC PRIMER for any other application.
- 3) TOKUSO CERAMIC PRIMER is intended for use only by licensed dental care professionals.
- 4) DO NOT use this product if the safety seals are broken or if the contents have been tampered with.
- 5) Always determine whether your patient is allergic or hypersensitive to methacrylic monomers. DO NOT use TOKUSO CERAMIC PRIMER if a patient is allergic to or hypersensitive to these monomers. If TOKUSO CERAMIC PRIMER causes an allergic reaction or hypersensitivity, discontinue its use immediately.
- 6) Use examination gloves (plastic, vinyl, latex) at all times while using TOKUSO CERAMIC PRIMER to avoid the possibility of allergic reactions from methacrylic monomers. If TOKUSO CERAMIC PRIMER comes in contact with the examination gloves, immediately dispose of the examination gloves and wash hands thoroughly with water.
- 7) AVOID CONTACT WITH EYES. In case of contact with eyes, thoroughly flush eyes with water and immediately contact a physician.
- 8) In the event that TOKUSO CERAMIC PRIMER contacts the skin, clean the skin immediately with an alcohol soaked cotton swab or gauze.
- 9) TOKUSO CERAMIC PRIMER should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
- 10) To avoid cross infection, DO NOT reuse mini sponges. Also clean the mixing well with alcohol after use.
- 11) DO NOT leave TOKUSO CERAMIC PRIMER within the reach of patients and children.
- 12) AVOID open flames when using or disposing primers.

### STORAGE

- 1) Store at 0-25°C (32-77°F).
- 2) DO NOT use TOKUSO CERAMIC PRIMER after the indicated date of expiration.

### DISPOSAL

The remaining liquid should be absorbed in an inert material such as gauze or cotton, and should be disposed of in accordance with local regulations.

\*The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure the product is suitable for an appropriate application before use. Specifications are subjected to change without notification.

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