ENGLISH

Carefully read all information, contraindications, precautions and notes before using PALFIQUE ESTELITE LV.

■CONTENTS:

PALFIQUE ESTELITE LV

- Paste: 3g syringe
- Syringe Tip: 15 pieces

Available Type and Shade

Type	Shade
High-Flow	A1, A2, A3
Medium-Flow	A1,A2,A3,A3.5
Low-Flow	A1, A2, A3

Please check with local distributor for available shades and types.

REFILL

- Syringe Tip (50 pieces/box)

■PRODUCT DESCRIPTION AND GENERAL INFORMATION

*PALFIQUE ESTELITE LV is a low viscosity, light cured submicron filled composite resin suitable only for the specific clinical applications listed below.
 *Direct placement of PALFIQUE ESTELITE LV to a cavity makes handling easy.
 *PALFIQUE ESTELITE LV comes in three viscosities, High-Flow, Medium-Flow and Low-Flow.

■CLINICAL APPLICATIONS

PALFIQUE ESTELITE LV may be used in the following clinical applications:

- *Restoration of class-V cavities
- *Restoration of deciduous teeth
- *Restoration of incipient caries
- *Restoration of small class-I, class-II and class-III cavities
- *Restoration of small cavities or tunnel preparation
- *Basing before composite resin restoration
- *Lining of class-V cavities prior to restoration with other composite resins
- *Contour correction after composite resin filling
- *Repair of fractured porcelain
- *Repair of small fractured composite resin
- *Blocking of undercut

■CONTRAINDICATIONS

*PALFIQUE ESTELITE LV contains methacrylic monomers. PALFIQUE ESTELITE LV is contraindicated in patients allergic to or hypersensitive to methacrylic and related monomers and should NOT be used in such patients.

■PRECAUTIONS

- 1) Use PALFIQUE ESTELITE LV only as directed in these instructions.
- 2) PALFIQUE ESTELITE LV is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor is it suitable for use by non-dental care professionals.
- 3) DO NOT use PALFIQUE ESTELITE LV for any purpose other than those listed in these instructions.
- 4) DO NOT use PALFIQUE ESTELITE LV if the safety seals are broken or appear to have been tampered with.
- 5) If patient experiences an allergic reaction or sensitivity to PALFIQUE ESTELITE LV, such as dermatitis eruption, discontinue its use immediately.
- 6) The use of examination gloves (plastic, vinyl, latex) is recommended at all times to avoid the possibility of allergic reactions from methacrylic monomers. If PALFIQUE ESTELITE LV comes in contact with the examination gloves, immediately dispose of the examination gloves and wash hands thoroughly with water.
- 7) AVOID CONTACT WITH EYES. If PALFIQUE ESTELITE LV comes in contact with eyes, immediately flush eyes with water and seek medical attention.
- 8) When using a light-curing unit, protective eye shields, glasses or goggles should be worn.
- 9) DO NOT mix PALFIQUE ESTELITE LV with other brands of composite resin to avoid incomplete cure or air bubble contamination. To avoid air bubble contamination, DO NOT mix PALFIQUE ESTELITE LV with another shade of the paste.
- 10) DO NOT use eugenol materials with PALFIQUE ESTELITE LV.
- 11) In the event that PALFIQUE ESTELITE LV contacts the skin, clean the skin immediately with an alcohol soaked cotton swab.
- 12) Protect pulp with a lining material, if the pulp is exposed.
- 13) PALFIQUE ESTELITE LV should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
- 14) DO NOT apply unnecessary force to the syringes, especially immediately after taking out of a refrigerator.
- 15) DO NOT leave PALFIQUE ESTELITE LV unnecessarily within the reach of patients and children to avoid being swallowed by mistake.
- 16) Clean the filling instruments with alcohol after use.
- 17) Attach the cap to the cartridge after each use; hold the nozzle and place the cap onto it. Turn the cap clockwise until it is securely locked.
- 18) Replace the syringe tip after each use.

■NOTES FOR USE

*Be sure to light cure PALFIQUE ESTELITE LV extraorally and check the time needed for complete hardening of PALFIQUE ESTELITE LV with your light curing unit before performing the bonding procedure.
 *PALFIQUE ESTELITE LV should NEVER be intermixed with other brands of composite resin, but can be layered with other brands of composite resin.

■STORAGE

- 1) Store PALFIQUE ESTELITE LV at a temperatures between 0-25°C (32-77°F).
- 2) AVOID direct exposure of PALFIQUE ESTELITE LV to light and heat.
- 3) DO NOT use PALFIQUE ESTEQUITE LV after the indicated date of expiration printed on the syringe and/or package.

■DISPOSAL

Safely dispose of excess PALFIQUE ESTELITE LV by extruding unused product from syringe with instruments and light cure before disposal.

■ATTACHING OF SYRINGE TIP

- 1) Hold the nozzle and turn the cap counterclockwise to remove. (①)
- 2) Hold the nozzle and place the Syringe Tip onto it. Turn the Syringe Tip clockwise until it is securely locked. (②)
- 3) To change the direction of Syringe Tip after attaching to the nozzle, turn the nozzle only; do not turn the entire Syringe Tip. (③)

- *Syringe Tips are also sold separately.
- *Use Gloves when attaching Syringe Tips.

■CLINICAL PROCEDURES

1. Clean the tooth surface using conventional methods.
2. Select shade.
3. Isolate.
4. Prepare cavity, protect pulp if necessary.

If pulpal protection is required, use a lining material which does not contain eugenol. Eugenol based products will inhibit curing of the bonding agent and/or composite resin.

5. Apply the bonding agent according to its manufacturer's instructions.
6. Filling and contouring
- *Cavities can be directly filled from the syringe, or indirectly filled with an instrument after extruding the paste on the mixing pad.
- * (for Low-Flow only) If it is necessary to use hand instrument after direct placement, use a light sweeping motion from the center of the cavity toward the cavity wall.
- *After extruding the paste, wipe the nozzle with tissue paper, etc. and replace the cap immediately.
7. Light cure
- *Irradiate by light for more than 30 seconds to ensure complete cure.

(Note)
 Be sure to check the light curing unit output. Refer to the instructions of high-powered light irradiation units such as Plasma irradiation units (with a Xenon lamp) before use.
 *Deep cavities should be filled incrementally.
 *If other brands of composite resins are layered over the cured PALFIQUE ESTELITE LV, follow their instructions.
 8. Final adjustment (polish the restoration).
 (Clinical cases for porcelains/cured composite resins)
 *Be sure to pretreat the surfaces of porcelains/cured composite resins with a silane coupling reagent BEFORE performing bonding procedure.

The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure the product is suitable for an appropriate application before use.
 Specifications are subjected to change without notification.

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Type	Farbe
High-Flow	A1, A2, A3
Medium-Flow	A1,A2,A3,A3.5
Low-Flow	A1, A2, A3

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TOKUYAMA DENTAL CORP.

MATERIAL SAFETY DATA SHEET

(According to 91/155/EEC and OSHA regulation 29 CFR 1910. 1200(g))

1. Identification of substance

Product name	PALFIQUE ESTELITE LV					
Description of Product	Flowable Composite Resin, For dental professionals only.					
Manufacturer	Tokuyama Dental Corporation 38-9, Taitou 1-chome, Taitou-ku, Tokyo, Japan					
EC Representative (Emergency Contact in EU)	Tokuyama Europe GmbH Oststrasse 10, 40211 Düsseldorf, Germany Tel:+49 211 1754480, Fax:+49 211 357379					
North/South America Middle East	J. MORITA USA, Inc. 9 Mason, Irvine, CA 92618, U. S. A. Tel: (949) 581 9600 Fax: (949) 465 1095 Tel: (800) 831 3222 Email: info@jmoritausa.com					

2. Composition / Data on components

Ingredients	%			Cas.No.	Symbols	R-phrases	OSHA PEL
	High Flow	Medium Flow	Low Flow				
Mixture of Silica-Zirconia and Silica-Titania fillers	68	68	65	—	—	—	N/E
Composition of Methacrylates	32	32	35	—	—	—	N/E

(No dangerous components)

3. Hazard identification

Repeated prolonged contact may cause skin inflammation.

4. First aid measures

After skin contact	Wash with soap and plenty of water.
After eye contact	Wash immediately with plenty of water. Get medical attention.
After swallowing	Induce vomiting immediately. Get medical attention.

5. Fire fighting measures

Suitable extinguishing media	Chemical foam, carbon dioxide or dry chemical
Media not to be used	None
Combustion products and resulting gases	None
Special precaution firefighters	Use protective clothing and a self-contained breathing apparatus.