


**1 Dispensing**Regular  
(for Crown)Small  
(for Inlay)**2 Mixing**


\*No need to split powder mound.

**4 Seating****3 Applying****5 Removing Excess**

\*Please see back for details.

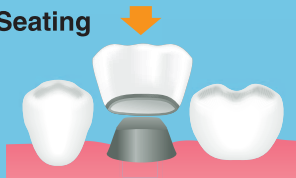


**a** Timely Excess Cement Removal

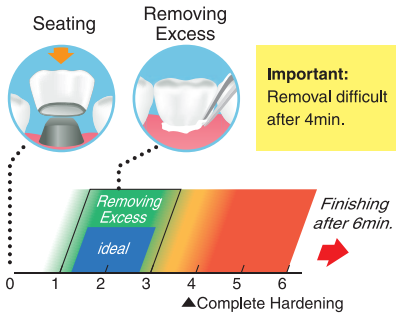

**2 min.** is ideal for Removal.

**b** Removal of Excess Cement

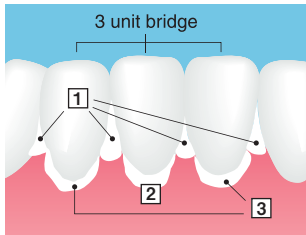

**Step1:** Inter Proximal area

**Step2:** Pontic Base

**Step3:** Remaining margin

**c** Medicaments may inhibit set.



e.g. Eugenol, iodine, etc.  
Please see instructions.

**d** Cast Post to be prepared 20 min plus after seating to secure proper adhesion.



Please see instructions.