Adhesive Resin Cement
Adhäsiv-Kunststoffzement
Ciment résine adhésif
Cemento resina adesivo
Cemento de resina

TOKUYAMA M-Bond

ENGLISH

Read all Information, Precautions and Notes before using M-BOND.

Contents

Mini Kit:			
Powder (Clear)	3g	Powder (Clear)	1g
Powder (Ivory)	3g	Powder (Ivory)	1g
Liquid	8mL	Liquid	3mL
Primer A	3mL	Primer A	1mL
Primer B	3mL	Primer B	1mL
Mixing Dish	1	Mixing Dish	1
Powder Measuring Spoon	1	Powder Measuring Spoon	1
Spatula	1	Spatula	1
Brush Handle	2	Brush Handle	2
Disposable Brush Tip	50	Disposable Brush Tip	50
Disposable Brush Tip (for Brush-on)	10	Disposable Brush Tip (for Brush-on)	10

Powder (Clear)	3g	Liquid	8mL
Powder (Ivory)	3g	Primer A	3mL
Powder (Dentin)	3g	Primer B	3mL
Powder (Opaque ivory (radiopaque))	4g	Disposable Brush Tip	100
Powder (Opaque dentin (radiopaque))	4g		
Disposable Brush Tip (for Brush-on)	50 (with Brush Handle x 1)		

- Features**
- PMMA type resin cement
 - High bonding strength
 - Excellent adhesive durability
 - Extended working time, 1 minute 40 seconds at room temperature
 - No rinse needed thanks to self-etching priming system

Ingredients

Description	Principal ingredients
Primer A	phosphoric acid monomer / acetone
Primer B	water / acetone / borate catalyst
Powder	PMMA / BPO
Liquid	MMA/ MAC-10/ methacrylate monomers / amine

Clinical Usage

Applications	Mixing method	Brush-on method
Bonding of metal inlay, onlay and crowns	A	A
Bonding of cast/prefabricated post	A	A
Bonding of adhesive/Maryland bridge	A	A
Bonding of adhesive splint	A	A
Fixation of loose tooth	N/A	A
Temporary repair of chipped tooth	N/A	A
Repair of fractured tooth	N/A	A
Bonding of orthodontic bracket/ retainer	N/A	A
	A: Available <p>N/A: Not Available</p>	

CONTRAINDICATIONS

M-BOND contains methacrylic monomers. M-BOND is contraindicated in patients allergic to or hypersensitive to methacrylic and related monomers and should NOT be used in such patients.

Clinical Application

This consists of three sections Basic Procedure, Pretreatment of Tooth, Pretreatment of Prosthesis. See each section for details.

Basic Procedure

- Pretreatment of prosthesis**
 - Refer to “Pretreatment of Prosthesis”.
- Cleaning of Cavity/Abutment**
 - Remove all contamination from the tooth surface, i.e. plaque, etc., with a brush or ultrasonic scaler.
 - Mechanically remove temporary sealer, temporary cement, etc. by ultrasonic scaler, etc. if it sticks firmly.
- Pretreatment of Tooth Surface**
 - Refer to “Pretreatment of tooth surface”.
- Mixing and Applying M-BOND Powder & Liquid**
 - Mixing method
 - “Regular” Mix: Dispense 3 drops of Liquid and 1 full scoop of powder, “Regular” side into Mixing Dish, and mix with Spatula.
 - “Small” Mix: Dispense 2 drops of Liquid and 1 full scoop of powder, “Small” side into Mixing Dish and mix.
 - Apply the mixture to the surface to be bonded before the mix gets stringy.
 - To avoid unexpected hardening of M-BOND with Primer, DO NOT apply the mixture to the tooth surface. Especially in the case of bonding posts, NEVER place the mixture in the root canal. Be sure to apply the mixture to the post and then set into the root canal.
 - Brush-on method
 - Place Powder and Liquid in separate wells of the Mixing Dish.
 - Immerse the Brush tip fully in the Liquid and touch the Powder with the Brush Tip to create mixture of cement on it.
 - Place the mixture of cement on the surface to be bonded.
- Prosthesis Seating, Pressing and Removal of Excess Cement**
 - Seat the prosthesis, press it to ensure the proper bond.
 - Remove the excess cement IMMEDIATELY.
- Curing and Finishing**
 - Allow M-BOND to cure while applying pressure on the prosthesis and allow to set 4 minutes.
 - 4 minutes after seating the prosthesis, the cement will be completely hard.
 - Follow standard procedures to polish and check the patient’s bite. **Be sure to perform occlusal adjustment.**

Pretreatment of Tooth (complete treatment : no rinsing necessary)

- Cleaning**
 - [Fixation of a Loose Tooth]
 - Clean tooth surface to be bonded thoroughly of all contaminants.
 - Perform scaling in advance if tartar and/or precipitate (plaque, biofilm etc.) are observed.
 - *In the case of treatment with phosphoric acid etching agent, rinse and dry then treat with Primers.

- [Bonding of Prosthesis / Restoration]
 - Remove any of the following factors that inhibit curing by thoroughly cleaning the tooth surface with alcohol, citric acid, EDTA, or phosphoric acid for 2-3 seconds before pretreatment of tooth surface:

- Packungsinhalt**
 - Normal-Packung
 - 1) temporary sealer, temporary cement, temporary crown
 - 2) silicone oil of fit check material
 - 3) oil mist from handpiece
 - 4) saliva, blood and exudate

- Apply Primer A / B**
 - Place one drop each of Primer A and B into the Mixing Dish.
 - Mix thoroughly and evenly with the Brush.
 - Immediately apply mixed Primer onto the entire tooth surface.
 - Allow Primer to sit for 30 seconds.

- Dry**
 - Use a weak-medium air stream for 5 – 10 seconds (**Do Not Rinse**).

Pretreatment of Prosthesis

Prosthesis made of metal (non-precious and precious), resins and ceramics. The preparation of the respective surfaces prior to treatment is as follows:

Protection of Prosthesis Surface

Protect the prosthesis surface Not to be bonded, with metal separator, paraffin film, Vaseline, etc., if necessary.

Non-precious Metal Alloys

- Clean prosthesis surface fully and alumina-sandblast.
- Ultra-sonically clean the prosthesis, allow to dry.
- *Sandblasting should be done with 30 to 50 µm alumina beads.
- *If unable to sandblast the prosthesis, roughen the surface with a bur, diamond point, etc.

Instruction for use
Gebrauchsinformation
Notice d'utilisation
Istruzioni per l'uso
Instrucciones de uso

CE

REF RC03

*Reapply surface protection again if unexpected removal of protection film occurs.

Precious Metal Alloys

- Alumina-sandblast, clean ultrasonically, and dry all in the same manner non-precious metal alloys are treated. After these steps are completed, treat metal surface to be bonded with METALITTE (sold separately) or tiplaste.
- *If unable to sandblast the prosthesis, roughen the surface with a bur, diamond point, etc.
- *Do not allow saliva or blood to come in contact with or apply any etching material to the prosthesis surface that has been treated with METALITTE or ti-plaste!
- *If the treated surface is contaminated by saliva or blood, clean the surface ultra-sonically to remove all contamination.
- *When using a primer for precious metals to replace tin-plating, such as METAL-ITE, be sure to refer to the instruction manual of the product being used.
- *Reapply surface protection if unexpected removal of protection film occurs.

Ceramics

- *Be sure to read Precautions 1).
- When treating porcelain, castable ceramics, machinable ceramics, etc., roughen the prosthesis to create a fresh bonding surface, clean ultrasonically and allow to dry completely.
- *Silane treatment; mix and apply TOKUSO CERAMIC PRIMER (sold separately) for 10 seconds, gently blow with air.

Resins

Composite resins for crown & bridge: treat surfaces the same as stated above under Ceramics.

Working Time

Working time of M-BOND is 1 minute 40 seconds (until the mixture becomes stringy) at room temperature (25 °C / 77 °F).

Precautions

- When using M-BOND for cementing ceramic restorations, be sure to confirm whether or not the restorations are treated with METALITE or Zirconia. DO NOT APPLY TOKUSO CERAMIC PRIMER for Alumina and/or Zirconia restorations. This will adversely affect optimum cementation of these restorations. For Alumina and/or Zirconia restorations, simply roughen restoration surfaces by using a sandblaster or diamond bur before cementing.
- M-BOND is designed for sale to licensed dental care professionals only. It is not designed for sale to nor is it suitable for use by non-dental care professionals.
- Use M-BOND as instructed. DO NOT use M-BOND for any application other than those noted in the Instructions.
- If patient experiences an allergic reaction or sensitivity to M-BOND, such as dermatitis eruption, discontinue its use immediately.
- Avoid contact of any M-BOND components with eyes. If contact with eyes occurs, thoroughly flush eyes with water, and immediately contact a physician.
- If any of M-BOND components contacts skin or clothing, clean it off immediately with an alcohol soaked cotton gauze.
- Contact of any of M-BOND components with mucosal membrane may result in a whitening of the affected area due to protein coagulation by acid monomer (whitened color will disappear within 1-2 days). If contact with mucosal membrane occurs, immediately wipe the area away and thoroughly flush with water.
- Avoid ingestion of any of M-BOND components because it is acidic.
- If the cavity is immediately proximate to the pulp, pulp should be protected with a Glass Ionomer lining or calcium hydroxide. DO NOT USE EUGENOL MATERIALS to protect the pulp. Eugenol material will inhibit curing of M-BOND.
- Use examination gloves at all times while using M-BOND to avoid the possibility of allergic reactions from methacrylic monomers. When the examination gloves come in contact with any of M-BOND components, dispose of the examination gloves immediately and use new examination gloves.

- Avoid open flames when using M-Bond, since Liquid, Primer A and Primer B are flammable.
- To avoid cross infection, DO NOT reuse Disposable Brush Tips. Clean the Mixing Dish, Spatula, Brush Handle and Spoon.
- Open the Liquid and Primer carefully. The internal vapor pressure might splash the liquid.
- Remove excess cement IMMEDIATELY after pressing the prosthesis/restoration, since the hardened M-BOND is difficult to remove.
- M-BOND SHOULD NOT BE MIXED OR USED with other types or brands of resin cements.
- DO NOT line cavity with any type of cement before applying M-BOND.
- Primers are highly volatile. Be sure to ventilate the room and screw the caps tightly within 3 minutes of using.
- DO NOT remove protective cover from the Primer bottles (It is designed to dispense the Primers smoothly).
- Remove the polymerization inhibitors such as temporary sealers, temporary cements and oil from fit checking materials completely from the surface to be bonded.
- Do not use this product if the safety seals are broken or if the contents have been tampered with.
- It is recommended that rubber to be attached to the Mixing Dish during the clinical procedure. Clean the Mixing Dish immediately after use because cleaning of the Mixing Dish becomes difficult after the hardening of M-BOND. If hardening occurs, immerse the Mixing Dish with the adhered M-BOND in warm water and then clean it.
- Soften Brush Tip prior to use by immersing the tip fully in M-BOND Liquid while rubbing the Brush Tip on the bottom of the Mixing Dish.
- When using brush-on procedure, dispose of remaining Powder and Liquid after use.

Storage

- Before and After Opening: Store M-BOND under refrigeration at a temperature between 0-10 °C (32-50 °F). Long periods of storage at room temperature may result in incomplete polymerization of paste due to deterioration.
- DO NOT use M-BOND after the indicated date of expiration.
- DO NOT leave M-BOND within reach of patients or children.

Disposal

To safely dispose of M-BOND components, remove unused product from the bottles, soak the liquid in tissue paper and dispose of it in accordance with local regulations.

*The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure the product is suitable for an appropriate application before use.
Specifications are subjected to change without notification.

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Tokuyama Dental Corporation