Standard (14)

Bleach (2)

Opalescent (3)

Opaque (1)

Shade Matching Guide

E =Excellent

V =Very Good

G =Good

F =Fair

- =Poor

for Class V preparations

ESTELITE Σ QUICK Basic Shade	Tooth shade ^{a)}														
	←light												dark→		
	B1	A1	B2	D2	A2	C1	C2	D4	A3	D3	B3	A3.5	B4	C3	A4
A1	V	E	V	G	V	F	-	-	-	-	-	-	-	-	-
A2	G	V	V	V	E	V	F	-	F	F	-	-	-	-	-
A3	-	-	F	G	V	V	V	G	E	G	G	F	-	-	-
A3.5	-	-	-	-	-	-	F	F	V	V	V	E	V	G	-

for Class III preparations


ESTELITE Σ QUICK Opalescent Shade	Tooth shade ^{a)}														
	←light												dark→		
	B1	A1	B2	D2	A2	C1	C2	D4	A3	D3	B3	A3.5	B4	C3	A4
OA1	V	E	F	F	F	F	-	-	-	-	-	-	-	-	-
OA2	-	F	V	V	E	G	F	F	F	-	-	-	-	-	-
OA3	-	-	-	F	G	F	F	F	E	V	F	V	-	-	-

a) Above tooth shades are selected according to the VITA Shade Guide. VITA is a registered trademark of VITA Zahnfabrik H. Rauter GmbH & Co.KG.

Clinical Procedure Guide

- Read instruction before operation.
- Vor der Anwendung die Gebrauchsanweisung lesen.
- Leggere le istruzioni prima dell'uso.
- Veuillez vous référer au mode d'emploi avant utilisation.
- Lea las instrucciones antes del uso.
- Lees de aanwijzingen voor gebruik.

TOKUYAMA BOND FORCE* Single Component, Self-Etching, Light-Cured Dental Adhesive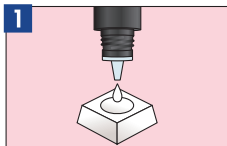

Dispense


Apply BOND FORCE


Then rub the margins

20
plus
sec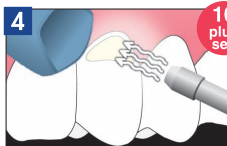

Apply weak air (5sec) → Strong air (5+sec)**

10
plus
sec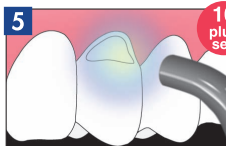

Light cure

10
plus
sec


*TOKUYAMA BOND FORCE bonds to uncut enamel without phosphoric acid etching. You don't need to etch the prepared cavity margin before applying TOKUYAMA BOND FORCE.

**Note: If accidental spattering of the material occurs, it may cause the tissue to whiten or a possible allergic reaction. To prevent spattering, refer to the instruction sheet.


***See the table of IFU.

ESTELITE Σ QUICK

Universal sub-micron filled quick curing composite resin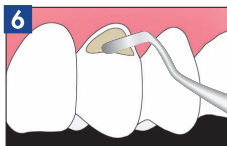


Fill & Sculpt

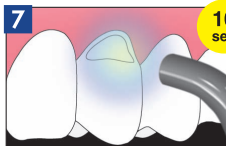Light cure

10
sec

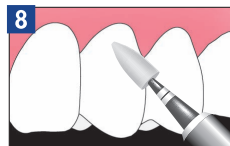Finish & Polish

NOTICE FOR USE OF THE ESTELITE Σ QUICK SHADE GUIDE

- Keep the ESTELITE Σ QUICK Shade Guide away from direct sunlight to prevent a change in color.
- Avoid contact between the ESTELITE Σ QUICK Shade Guide and bonding agents, etc., because the surface of the ESTELITE Σ QUICK Shade Guide will dissolve if it comes into contact with methacrylic monomers or solvents.

The ESTELITE Σ QUICK Shade Guide is composed of the following materials:

Shade piece: Polystyrene

Base plate: ABS (Acrylonitrile-Butadiene-Styrene) resin

- Refer to the following table when cleaning the ESTELITE Σ QUICK Shade Guide.

	Cleaning method
Correct	<ul style="list-style-type: none">wash with soap and water ultrasonic cleaning while immersed in water immerse in a 3% glutaraldehyde solution wipe with a cotton swab soaked in an alcohol (80%) solution
Incorrect	<ul style="list-style-type: none">autoclave¹⁾ dry heat sterilization¹⁾ immerse in an alcohol (80%) solution²⁾ immerse in a sodium hypochlorite solution³⁾ immerse in a hydrogen peroxide solution³⁾

- Shade piece and base plate will melt.
- Base plate will whiten and crack. Shade indications and the logo will fade out.
- Shade piece might be decolorized.

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