SOFRELINER TOUGH SHAPE ADJUSTMENT POINT SOFRELINER TOUGH FINISHING POINT SOFRELINER TOUGH DIAMOND ROUND BUR

ENGLISH

SOFRELINER TOUGH SHAPE ADJUSTMENT POINT (White)
SOFRELINER TOUGH FINISHING POINT (Brown)
SOFRELINER TOUGH DIAMOND ROUND BUR

PRODUCT DESCRIPTION

These points are only used for SOFRELINER TOUGH.

Materials	shapes	Stages
SHAPE ADJUSTMENT POINT (White)	Round end taper	Shape Adjustment
FINISHING POINT (Brown)	Round end taper	Finishing
DIAMOND ROUND BUR	Round	Inner Surface Adjustment

CLINICAL APPLICATION

This is intended to use together with handpiece or any other dental driving units for straight type shank.

PRECAUTIONS

Materials	Rotating speed and precaution
SHAPE ADJUSTMENT POINT (White)	Correct the form by sliding the point lightly on the surface under 15,000 rpm.
FINISHING POINT (Brown)	Finish by sliding the point lightly on the surface under 4,000rpm.
DIAMOND ROUND BUR	Use under 15,000rpm.

- * Check if the shank is correctly set to the handpiece.
- * Check if the point rotates correctly without any vibration.
- * The durability of the point goes down if it is used more than the rotating speed as indicated of each point.
- * Do not use it at an improper angle.
- * Avoid contact of the point with body or clothes.

NOTES FOR USE

1) Caution

- * Do not use exceeding the rotating speed as indicated.
- * Do not use if there are any damage or deformation found.
- * Use eye protective shield such as glasses.
- * Avoid inhalation of dust by using a dust collector or generally accepted dustfree mask.
- * Use only as instructed herein. Do not use for any other application.
- * For use only by licensed dental professionals.

2) Other Caution

This information which is contained in this document is based on available data.

Information in this document are subject to change without notification.

STORAGE

- * Keep the point away from non-dental personnels.

Packing

SOFRELINER TOUGH SHAPE ADJUSTMENT POINT (White) × 3
SOFRELINER TOUGH FINISHING POINT (Brown) × 6
SOFRELINER TOUGH DIAMOND ROUND BUR × 1

- *The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure the product is suitable for an appropriate application before use. Specifications are subject to change without notification.

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Tokuyama Dental Corporation
Tokuyama Europe GmbH
J. MORITA USA, Inc.
9 Mason, Irvine, CA 92618, U.S.A.